

Dr. Ivan A. Serdar

San Francisco, CA



- 12/02/08** Pumice, rinse, one minute etch with 37% phosphoric acid, rinse, blot, five minutes MI Paste.
- 01/07/09** Pumice, rinse, one minute etch with 37% phosphoric acid, rinse, blot, five minutes MI Paste.
- 01/26/09** Pumice, rinse, one minute etch with 37% phosphoric acid, rinse, blot, five minutes MI Paste.
- 02/25/09** Pumice, rinse, one minute etch with 37% phosphoric acid, rinse, blot, five minutes MI Paste.
- 04/01/09** Pumice, rinse, one minute etch with 37% phosphoric acid, rinse, blot, five minutes MI Paste.
- 04/29/09** Pumice, rinse, one minute etch with 37% phosphoric acid, rinse, blot, five minutes MI Paste.

Patient used trays at-home for the duration of this treatment and placed MI Paste in her trays for five minutes, twice a day. Patient also used regular strength Whitestrips¹ at home for three weeks.

Dr. Ronald Rubin & Dr. Cindy Pong

Cincinnati, OH

Hypoplastic Enamel



- Two minute etch with 37% phosphoric acid (blue gel)
- Heavy pumice
- Dispense tube of MI Paste or MI Paste Plus. Patient apply/rub MI Paste with finger to affected areas for 10 seconds, daily at bedtime; re-evaluate in two weeks
- Repeat sequence as needed for desired result
- Treatment was over the course of one month: 9/3/2008-9/29/2008

Dr. Stephanie Benton

Grand Rapids, MI



- Dr. Benton's teeth were etched twice for 20 second intervals
- No pumice was used
- She placed MI Paste immediately on her teeth following etching and wore trays at night time only
- She had immediate results over the first week
- She continued to wear the trays for two and a half weeks at night time
- "Very happy with the results!"

Dr. Dan Jenkins

Chino, CA

Case One



- Completed three in-office MI Paste treatments
- Applied MI Paste each night for for weeks between treatments

Case Two



- Completed three in-office MI Paste treatments
- Applied MI Paste each night for four weeks between treatments